MEDISANA®

DE **Personenwaage PSL**

GB **Personal scales PSL**

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IT **Bilancia pesapersona PSL**

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NL **Personenweegschaal PSL**

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SE **Personvåg PSL**

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Lot number

Manufacturer

Safety Information

- Any treatment or diet for people who are underweight or overweight requires the qualified advice of a professional (doctor, dietician). The values determined with the scales may be source of useful support.

- **Risk of tipping over!**

Do not place the scale on an uneven surface.

Never step onto a corner of the scales.

When weighing yourself, do not stand on one side or the edge of the scales.

- **Risk of slipping!**

Do not stand on the scale with wet feet.

Do not stand on the scale wearing socks.

- Use the device only according to its intended purpose as specified in the instruction manual.
- The warranty will be invalidated if the device is used for purposes other than those for which it is intended.
- The scale has been manufactured for domestic use. It is not suitable for commercial use in hospitals or other medical institutions.
- Do not use the device if it is not working correctly, if it has been dropped or has fallen into water or has been damaged.
- The scales have a measuring range up to 180 kg, 396 lb/28.8 st. Do not overload the scales.

- Place the scales on a firm, level base. Soft, uneven surfaces are unsuitable for the measuring process and lead to incorrect results.

- Do not use the scale in a location where extremely high temperatures or humidity may occur.

- Keep the scale away from water.

- Handle the scale with care. The scale must not be subjected to impacts or vibration. Do not drop the scale.

- Do not attempt to dismantle the scale, otherwise the warranty will be invalidated. This unit contains no user-maintainable or user-replaceable parts apart from the battery.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.
- If the scale is not going to be used for a long period, please remove the batteries.

Thank you very much

Thank you for your confidence in us and congratulations!
By purchasing the personal scales **PSL**, you have acquired a quality product from **MEDISANA**.

This unit is intended for weighing people.

To ensure that your **MEDISANA** personal scales **PSL** provide successful results for a long time, we recommend that you read carefully through the following notes on their use and care.

2.1 Items supplied and packaging

Please check first of all that the unit is complete and is not damaged in any way. If in doubt, do not use the appliance. Send it to a service point.

The following parts are included as standard:

- 1 **MEDISANA PSL** Personal scales
- 1 Type CR2032 3V lithium coin cell battery
- 1 Instructions for use

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**

Please ensure that polybag packing is kept away from reach of children! Risk of suffocation!

2.2 Performance features

- Top quality safety glass
- Top quality sensor technology
- Measuring range 5 kg to 180 kg or 11 lb to 396 lb/28.8 st
- 100 g, 0.2 lb or 1/4 lb graduation
- Switchable between kg, lb and st
- Direct measurement of body weight
- Easy to use
- Clear LCD display
- Includes 1 type CR2032 3V battery

2.3 Prerequisites for correct measuring results

Reliable measurements can only be achieved under certain prerequisites:

- Always weigh yourself under the same conditions.
- Always weigh yourself at the same time of day.
- Stand up straight and stand still.
- Weigh yourself preferably in the morning, after showering or bathing, and with dry feet.

It is possible to obtain a reliable indication of body weight development from the results if measurements are carried out consistently.

3.1 Insert/replace batteries

One button cell batterie (type CR2032) is already inserted in your new scale. The insulating strip must be pulled out of the battery compartment ❶ on the underside of the unit before the scale can be operated. Replace the batterie if the battery change symbol "Lo" appears in the display ❷ or if nothing appears in the display when the device is switched on. To do this, open the battery compartment beneath the device and insert one cell batterie (type CR2032). Place the battery in the empty compartment (not the compartment with the red sticker)! Please note polarity (positive pole upwards; facing towards the lid). Close the battery compartment again.

WARNING BATTERY SAFETY INFORMATION

- Do not disassemble batteries!
- Remove discharged batteries from the device immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with skin, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge these batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

3.2 Weighing

Place the scales on a hard, level base. This ensures accurate measuring results and is the requirement for safe weighing without risk of injury.

1. If you have moved the scales, this may cause the display to be activated. If anything other than **"0.0 kg"** is displayed, wait approx. 30 seconds until the scales switch off automatically.
2. Switch the scales on by briefly tapping the standing area **④** with a foot.
3. **"8888"** will appear briefly on the display **③** followed by **"0.0 kg"**. The personal scales are ready to measure. Do not step onto the scales before **"0.0 kg"** is displayed!
4. Stand on the scales and remain still. The display moves while your weight is being calculated. When the measurement has stabilised, the display stands still. Your measured body weight, e.g. **65.1** (65 kg 100 g), will be displayed for several seconds. The scales then switch off automatically. Step down from the scales.

NOTE

The scales will switch off automatically to save batteries if you do not step onto the scales within 10 seconds.

3.3 Change the Unit of Weight

The measuring unit of the **MEDISANA** personal scales **PSL** may be changed over. Turn the scales over and select the desired measuring unit KG, LB or ST using the black changeover switch **②**.

3.4 Error messages

The following error messages may appear in the display:

- Lo** Replace the battery.
- Err** The scale is overloaded.

If your personal scales do not function to your satisfaction, check the following before contacting a service centre.

- Check whether the batteries have been inserted correctly.
- Check whether the correct unit of weight has been selected.
- Check whether the scales are freestanding on a solid and even surface. They must not be touching a wall or any other object.
- Weigh yourself again and make sure that you only step on the scales when **"0.0 kg"** has appeared in the display.

4.1 Care and maintenance

- Remove the battery before cleaning the unit.
- Never use aggressive cleaning agents or stiff brushes.
- Clean the scale using a soft and lightly moistened cloth. Never use abrasive cleaning agents or alcohol.
- Do not allow any water to get into the unit. Do not use the unit again until it is completely dry.
- Remove the battery from the unit if it will be unused for a longish period. Otherwise there is a risk that the batteries might leak.
- Do not expose the unit to direct sunlight; protect it against dirt and moisture.

4.2 Disposal

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the battery before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

4.3 Specifications

Name and model	:	MEDISANA PSL Personal scales
Voltage supply	:	3 V= , 1 x 3 V lithium coin cell battery type CR2032
Measuring range	:	5 kg to 180 kg or 11 lbs to 396 lbs /28.8 st
Graduation	:	100 g, 0.2 lb or 1/4 lb
Autom. switch-off	:	after approx. 10 sec.
Dimensions (L x W x H)	:	approx. 32 cm x 31 cm x 2 cm
Weight	:	approx. 2.14 kg
Item number	:	40425
EAN number	:	4015588 40425 2

In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.

5.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear (batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA AG

Itterpark 7-9
40724 Hilden
Germany
Tel.: +49 (0) 2103 / 2007-60
Fax: +49 (0) 2103 / 2007-626
eMail: info@medisana.de
Internet: www.medisana.de

The service centre address is shown on the last page.

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ALERE Electronics AB