### DE Analoge Personenwaage PSD

- ❶ Einstellrad
- ❷ Anzeigeskala, max. 150 kg

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- ① Dial
- ② Display scale, max. 150 kg

#### Explanation of symbols:

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

#### LOT

LOT number

Manufacturer

## EN Safety Information

**Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.**

- Any treatment or diet for people who are underweight or overweight requires the qualified advice of a professional (doctor, dietician). The values determined with the scales may be source of useful support.
- Risk of tipping over! Do not place the item on an uneven surface. Never step onto a corner of the scales. When weighing yourself, do not stand on one side or the edge of the scales.
- Risk of slipping! Do not stand on the scale with wet feet. Do not stand on the scale wearing socks.
- Use the device only according to its intended purpose as specified in the instruction manual.
- The warranty will be invalidated if the device is used for purposes other than those for which it is intended.
- The scale has been manufactured for domestic use. It is not suitable for commercial use in hospitals or other medical institutions.
- Do not use the device if it is not working correctly, if it has been dropped or has fallen into water or has been damaged.
- The scales have a measuring range up to 150 kg. Do not overload the scales.
- Place the scales on a firm, level base. Soft, uneven surfaces are unsuitable for the measuring process and lead to incorrect results.
- Do not use the scale in a location where extremely high temperatures or humidity may occur.
- Keep the scale away from water.
- Handle the scale with care. The scale must not be subjected to impacts or vibration. Do not drop the scale.
- Stand carefully on the scale. Do not jump or hop onto the standing area. This could lead to defects of the weighing mechanism.
- Do not attempt to dismantle the scale, otherwise the warranty will be invalidated. This unit contains no user-maintainable or user-replaceable parts.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.

## Items supplied and packaging

Please check first of all that the unit is complete and is not damaged in any way. If in doubt, do not use the appliance. The following parts are included as standard:

- 1 medisana Analogue personal scales PSD
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

### WARNING

**Please ensure that polybag packing is kept away from reach of children! Risk of suffocation!**

## Prerequisites for correct measuring results

- Always weigh yourself under the same conditions.
- Always weigh yourself at the same time of day.
- Stand up straight and stand still.
- Weigh yourself preferably in the morning, after showering or bathing, and with dry feet.

It is possible to obtain a reliable indication of body weight development from the results if measurements are carried out consistently.

## Weighing

1. Place the scale on a stable, firm surface. All four feet must be touching the floor. Uneven and soft flooring affects measuring accuracy.
2. Always weigh yourself on the same scale, at the same place and at the same time. Weigh yourself without clothing, before meals, preferably in the morning before breakfast.
3. Stand with both feet on the scale. The tongue will point to your weight in kg.

## NOTE

If the scales have been transported or moved, you can calibrate them by tapping the surface lightly with your foot. Is the tongue not pointing to 0.0 kg, adjust it by turning the dial ② to left or right, until it points to 0.0 kg. Repeat this procedure.

## Care and maintenance

- Never use aggressive cleaning agents or stiff brushes.
- Use a mild household detergent and clean the surfaces only with a lightly moistened cloth.
- Do not allow any water to get into the unit. Do not immerse the device in water. Do not use the unit again until it is completely dry.
- Do not expose the unit to direct sunlight; protect it against dirt and moisture.

## Disposal

This product must not be disposed together with the domestic waste. Metal parts can be reused or recycled. Hand in a no longer required unit at a municipal or commercial collection point so that it can be disposed of in an environmentally acceptable manner.

Consult your municipal authority or your dealer for information about disposal.

## Specifications

**Name and model:** medisana Analogue personal scales PSD  
**Measuring range:** up to 150 kg  
**Graduation:** 500 g  
**Dimensions:** 29 \* 43 \* 8.5 cm  
**Item number:** 40461  
**EAN number.:** 4015588 40461 0

**In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.**

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

## Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for medisana products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**medisana GmbH**  
 Carl-Schurz-Str. 2  
 41460 NEUSS  
 GERMANY

The service centre address is shown on the attached leaflet.

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