BLUETOOTH®-Thermometer TM 735

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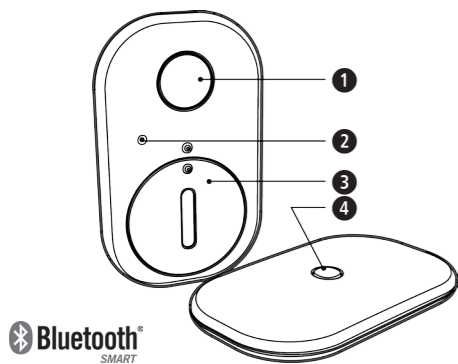
MEDISANA®

BLUETOOTH®-Thermometer TM 735

- 1 On-/Off-button
- 2 Function LED
- 3 Battery compartment
- 4 Temperature sensor

Assigned purpose:

The **MEDISANA Thermometer TM 735** is a wireless, battery-operated device to measure and monitor the body temperature on humans by measurement below the armpit.

The device transmits the measured results via **Bluetooth®** to the respective app on the smartphone of the user.

Contraindications:

The device is not suitable for use in emergency cases. If used on children below the age of 2 years, contact your doctor beforehand.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

ATTENTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Classification: Type BF applied part

LOT LOT number

Manufacturer

Date of manufacture

CE 0297

GB Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- Only use this device for the purpose for which it was developed and in the manner specified in the instructions for use. When using it for other purposes, the warranty expires.
- This device is not designed for commercial use.
- This device is only designated for interior use.
- Using this thermometer is no substitute for consulting a doctor.
- The measurement results provided by this device are not suitable to replace clinical resp. medical diagnoses.
- The thermometer and the patient must remain in a stable environment (at room temperature) for at least 5 minutes before measuring the temperature.
- This device is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- Children must be supervised to ensure that they do not play with the device.
- If a fault occurs or the unit is damaged, do not attempt to repair the unit yourself, as this will void the warranty. Contact your dealer and only have repairs carried out by authorized service partners. Do not use the thermometer if it is damaged.
- Do not open up the device. It contains small parts which might be swallowed by children.
- Do not use the thermometer in damp environments. Do not immerse the thermometer in water or other liquids. It is not watertight.
- Do not shake or tap the thermometer. Do not drop it.
- Only use the device in accordance with the operating conditions (see "Technical specifications"). The use under unsuitable conditions may lead to measurement errors and to the damage of the device.
- If the thermomter is not going to be used for a long period, remove the battery.
- Avoid the contact with pointed or sharp objects and do not disassemble or modify the device.
- Do not use the unit in environments with strong, electromagnetic radiation, highly flammable gases or nitrous oxide.
- Do not use the device together with MRI or CT equipment.
- Do not use too much pressure upon application of the thermometer. If the cover shows damages, do not use the device.
- During temperature monitoring, the sufficient skin contact of the device should be checked regularly in order to avoid measurement errors.
- High muscle or fat rates in the body may negatively affect the range of the wireless transmission.
- Sudden movements or insufficient battery power may lead to inaccurate measurement results.
- The thermometer must not be applied more than 24 hours to the same part of the skin. If you feel unwell during the application, remove the device and finish the measurement.
- Only use original parts from the manufacturer.
- The adhesive pads are assigned for single use only. Do not use them repeatedly and only use them in smooth and undamaged condition.
- Shave your armpit in advance, so that the area of skin which will be in contact with the patch is flat and smooth.

BATTERY SAFETY INFORMATION

- Do not disassemble batteries!
- Clean the battery and device contacts if necessary before putting in the batteries!
- Remove discharged batteries from the device immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Insert the battery correctly, observing the polarity!
- Keep the battery compartment well sealed!
- Remove the battery from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge batteries! **There is a danger of explosion!**
- Do not short circuit! **There is a danger of explosion!**
- Do not throw into a fire! **There is a danger of explosion!**
- Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

Scope of delivery

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre.

The following parts are included:

- 1 **MEDISANA Bluetooth®-Thermometer TM 735**
- 1 Box with 10 adhesive pads
- 1 button cell battery CR 2025, 3V
- 1 Battery lid opener
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

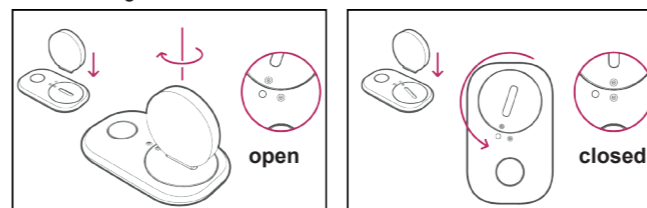
WARNING

Please ensure that polythene packing is kept away from the reach of children! Risk of suffocation!

GETTING STARTED - Removing / inserting batteries:

The **MEDISANA Bluetooth®-Thermometer TM 735** is directly ready to use, as the battery is already inserted into the device. You just need to remove the protection foil by pulling it out, so that the device is provided with electricity of the battery. If the battery loses performance after a certain time of usage, replace it with a new one. Therefore you need to remove the lid of the battery compartment 3 by turning it clockwise (e.g. with the help of the lid opener or a coin). Replace the battery with a new one of same type. Pay attention to insert it correctly („+“ - pole is visible). Close the lid again properly by inserting the lid and turning it in anti-clockwise direction.

Downloading the app

Download the app and install it on your smartphone. You may download the app for free from Google Play Store or Apple App Store.

Search for „**Medisana Thermo App**“.

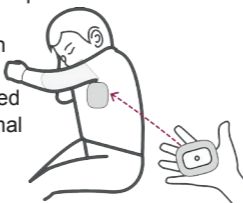
Establish a connection

Activate **Bluetooth®** on your iOS- or Android-device. Press and hold the On-/Off-button 1 for at least 2 seconds to switch on the thermometer. The function LED 2 will flash. Start the app on your smartphone and follow the instructions given in the app in order to establish a connection between the **MEDISANA Bluetooth®-Thermometer TM 735** and your smartphone and to synchronize the devices. If the connection has been established successfully, a temperature display will appear on the display of your smartphone. In order to avoid disturbances of the connection by phone calls, we recommend to activate the flight mode on your smartphone, but keep **Bluetooth®** also activated.

Measuring the body temperature

The thermometer and the patient must remain in a stable environment for at least 5 minutes before measuring the temperature. Clean your armpit and the surrounding area before starting a measurement. Take 1 adhesive pad, remove the protection foil and place the pad centered on the device (on the side where the battery compartment is located) - the temperature sensor needs to be in contact with the body. Place the device with the adhesive pad directly on your skin just below your armpit as illustrated.

Lower your arm and hold it lightly towards your body for at least 8 minutes. In order to keep a good connection between device and smartphone, avoid distances above **3 meters** between both devices. Also, the smartphone should be placed on the same side of your body as the thermometer. If a normal **Bluetooth®**- connection is established, the distance may be added up to a maximum of 10 meters. However, obstacles (like e.g. walls) may weaken the connection. The temperature display will appear on your smartphone (within the app) accordingly.

Monitoring the body temperature

To constantly monitor the body temperature, you may create user accounts within the app, which may be accessed through internet or network connection. With this you are able to log in the account from everywhere and monitor the measured temperature. Please pay attention to the instructions given by the app.

Functions of the On-/Off-button

The On-/Off-button 1 features the following functions:

- a. Press and hold for 2 seconds: Switches the device on. The function LED 2 flashes.
- b. Short press: Checks the device state. Depending on the state the function LED 2 will emit different signals: A single, short flash means, that the device has successfully established a connection to the app. Two flashes indicate, that there is no

connection to the app. No reaction of the LED means, that the device is off.

- c. Double press and hold button for 2 seconds: Switches the device off. The LED 2 lights up for 2 seconds, before the device will be switched off completely.

NOTE

If the device is not connected to the app for 60 minutes or is not in contact with the body, it will switch off automatically.

Cleaning and Disinfection

- To clean the thermometer, use a soft cloth slightly dampened with mild soapy water or 75% isopropyl alcohol. Do not let water get into the unit. Only use the unit again when completely dry.
- Never use aggressive cleaning agents, thinners, benzine or tough brushes.
- Remove the battery if you do not wish to use the unit for an extended period of time. Otherwise there is a danger that the battery may leak.
- If the thermometer is used as described in these instructions, no regular maintenance or calibration is required.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the battery before disposing of the device / unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your supplier for information about disposal.

Technical specifications

Name and model:	MEDISANA Bluetooth®-Thermometer TM 735
Power supply:	3V=, 1 x CR2025 Button cell battery
Measuring range:	25°C - 45°C (77°F - 113°F)
Accuracy of measurement:	± 0,05°C (0,09°F) at 35°C-38,5°C (95°F-101.3°F) ± 0,1°C (0,18°F) at <35°C (94.9°F) and >38,5°C (101.3°F)
Operating conditions:	+5°C - +40°C (+41°F - +104°F), rel. humidity 15 - 85 %; Air pressure 86 - 106 kPa
Storage conditions:	0°C - +50°C (+32°F - +122°F), rel. humidity 10 - 85 %; Air pressure 70 - 106 kPa
Compatibility:	iOS: Version 7.0 and newer. Android: Google-Android-Version 4.3 and newer. Bluetooth® 4.0 and newer.
Dimensions:	Approx. 51,9 x 31,6 x 6,5 mm
Weight:	Approx. 6 g (without battery)
Article No.:	76165
EAN No.:	4015588 76165 2

CE 0297

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under www.medisana.com

The CE-mark of this device refers to the EC-directive 93/42/EEC MDD. We hereby declare that the **MEDISANA Bluetooth®-Thermometer TM 735**, Art. 76165, complies with the fundamental requirements of the european RTTE Directive 1999/5/EC. You can request the complete EC Conformity Declaration from Medisana AG, Jagenbergstrasse 19, 41468 Neuss, Germany, or you can also download it from the Medisana homepage.

Electromagnetic compatibility: The device complies with the EN 60601-1-2 standard for electromagnetic compatibility. Please check the supplementary sheet for details on this measurement data.

Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

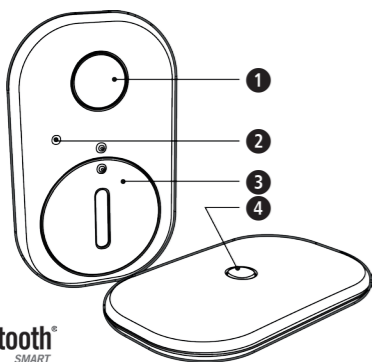
1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear (e.g. batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA AG

Jagenbergstraße 19
41468 NEUSS
Germany
E-Mail: info@medisana.de
Internet: www.medisana.com

The service centre address is shown on the attached leaflet.

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