# MEDISANA®

## VibrationBall

Trainingsvideo

Workout video

Art. 79510

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## IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

### Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

#### **WARNING**

These warning notes must be observed to prevent any injury to the user.

#### **CAUTION**

These notes must be observed to prevent any damage to the device.

#### **NOTE**

These notes give you useful additional information on the installation or operation.

Protection category II

**LOT**

Lot number

Manufacturer

### ***Intended use***

The **MEDISANA VibrationBall** is intended for a combined vibration-/pressure (by own weight) - massage application on adults to attain a relaxation deep down the muscle and to loosen tensions. Therefore, the suitable exercises for this device (see exercise instructions in the workout video or on the exercise card) have to be conducted.

### ***Contraindications***

- The **MEDISANA VibrationBall** should not be used if problems with the musculoskeletal system or with the organs are present. Check with your physician before using the device.
- Consult your doctor as well if:
  - a pregnancy, implants or pacemakers are present
  - you have circulatory disorders, wounds, bruises, inflammations or varicose veins.

### ***about the power supply***

- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Only insert the power supply unit into a socket when the appliance is switched off.
- To charge the integrated batteries, only use the power supply unit supplied for the device.
- Keep the power supply unit, cable and appliance well away from heat, hot surfaces, moisture and liquids.
- Do not touch the power supply unit when you are standing in water and always take hold of the plug with dry hands.
- Do not take hold of a device if it falls into water. Pull out the power plug and the power supply unit from the socket immediately.

- Connect the device for charging so that the power plug is freely accessible.
- Always switch off the appliance after use with the function button.
- To disconnect the unit from the power supply, always pull the plug out of the socket. Do not pull on the cable!
- Do not carry, pull or twist the device on the cable or the power supply unit.
- You must not continue to use the unit if the lead or the mains adaptor is damaged. For safety reasons, these parts may only be replaced by authorized service personnel. In order to prevent hazards, always send the unit in to the service centre.
- Ensure that the cable will not cause tripping. The cable must not be kinked, trapped or twisted.

### ***specific groups of people***

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children must be supervised to ensure that they do not play with the device.
- Do not use this unit to supplement or replace medical treatment. Chronic complaints and symptoms may be made worse.
- Do not use the appliance near organs or near any other sensitive areas.
- If you experience any pain or discomfort during the massage, stop immediately and consult your doctor.
- Before using the device, consult your doctor regarding any unexplained pain, if you are undergoing medical treatment or using medical equipment.

### ***before using the unit***

- Check the power supply unit, cable and device carefully for damage before each use or charging. Do not attempt to use a defective unit.
- Do not use the appliance if it, or the power cord show signs of damage, if it is not in perfect working order, or if it has been dropped or become damp. In order to prevent hazards, always send the unit in to the service centre.

### ***operating the unit***

**The device is not to be used for commercial use or medical purposes.**

**If you have health concerns, consult your doctor before using the VibrationBall.**

**Only use indoors!**

**Do not use the device in damp rooms (e.g. when bathing or showering).**

- The device must only be used for its intended purpose as described in the instruction manual.
- Using the unit for any other purpose invalidates the warranty.
- Only use the appliance in the proper place, as described in the exercise card.
- Do not use the appliance while you are asleep or when you are lying in bed.
- Do not use the appliance before going to sleep. The massage has a stimulating effect.
- Never use the appliance while driving a vehicle or operating a machine.
- The maximum operating time for one session with vibration is 15 minutes. After this time is exceeded, the **VibrationBall** automatically switches off.

- Let the motor cool-down for several minutes, before you switch on the **VibrationBall** again.
- Never leave the unit unattended if it is connected to the power supply (for charging the batteries).
- Do not allow the appliance to come into contact with pointed or sharp objects.
- Do not cover up the appliance when it is switched on. Never use it under blankets or cushions. There is a risk of fire, electric shock or injury.
- If applied to head, neck or upper back we recommend to switch off the vibration massage.

### ***maintenance and cleaning***

- The integrated batteries are not designated for exchange or removal by the end user. If problems with the power supply should occur, contact the service centre.
- You may only clean the device yourself. If the device malfunctions please do not repair it yourself as this leads to the automatic expiry of all warranty rights. Ask your specialist dealer and have repairs carried out by the service.
- Cleaning and user maintenance shall not be made by children without supervision.
- Do not immerse the device in water or other fluids.
- If, nonetheless, liquid should penetrate the device, switch it off immediately and unplug the mains cable if it was connected. Wait until the appliance has dried completely.

### Thank you very much

Thank you for your confidence in us and congratulations!  
 You have gained a high-quality product from **MEDISANA** with the purchase of this **MEDISANA VibrationBall**. To derive the most success and to enjoy your **MEDISANA VibrationBall** a very long time, we recommend that you read the following information on the use and care of the appliance carefully.

### 2.1 Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre.

The following parts are included:

- 1 **MEDISANA VibrationBall**
- 1 Power supply (for charging) with cable
- 1 Exercise card with the most important exercises
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

If you notice any transport damage during unpacking, please contact your dealer without delay.

### WARNING

**Please ensure that all packaging films are kept away from the reach of children! Risk of suffocation!**

### 2.2 How does the MEDISANA VibrationBall work?

The **MEDISANA VibrationBall** is a massage ball with vibration motor for the deeply efficacious vibration massage. It supports to prevent muscle tensions and to regenerate your muscles more quickly (e.g. after a intense workout). The extremely powerful motor provides a second-to-none vibration effect in 4 different intensities. The **MEDISANA VibrationBall** combines the deep vibration massage with a pressure massage (by your own weight), so that the tensed muscles will be effectively loosened.

### CAUTION

**Make sure that the maximum operating time of 15 minutes is not exceeded.**

### 3.1 Battery charging

Before first use the integrated batteries should be charged (lithium-ion batteries 7.4V, 2200mAh) completely. Therefore connect the included power supplier to a regular mains outlet and the other end of the cable to the connection for power supply ②. To charge the batteries fully, the charging time should be approx. 3 hours. The 3 LED-lights ① are showing the state of charge:

 = approx. 33 %     
  = approx. 66 %  
 = approx. 67 - 100 %

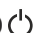
After the batteries have been charged fully, you may disconnect the power supply. With fully charged batteries, you can use the device for up to approx. 70 minutes (level 4).

### 3.2 Application with massage function

1. Press the function button ④ : Vibrating massage is switched on (intensity level 1).
2. Press the function button again: Intensity level increases.  
(Pressing function button on intensity level 4 turns off vibrating massage)

The **MEDISANA VibrationBall** provides 4 levels of massage intensity. The 4 LED lights ③ show the actual adjusted level:

 = Intensity level 1     
  = Intensity level 2  
 = Intensity level 3     
  = Intensity level 4

Please pay attention to the workout video and exercise card to train correctly. It is important to take your time during exercise - do it slowly and sensible. The maximum exercise time with vibration should not exceed 15 minutes. The unit will switch off automatically after approx. 15 minutes. If you want to switch it off before that moment, press and hold function button ④ .

### 3.3 Application without massage function

The **MEDISANA VibrationBall** may also be used without switching on the vibration massage function. Rolling it under your body with the pressure from your own body (see details how to exercise correctly in the workout video and on the exercise card) makes a pressure massage possible. This is suitable for more sensitive parts of your body, like e.g. the back and neck area. If the vibration massage is inconvenient, you may massage the respective parts of the body also without vibration.

### 3.4 Recommendations

We recommend to drink sufficient amounts of liquid (e.g. mineral water etc.) before and after your exercise with the **MEDISANA VibrationBall**. This supports the fluid balance positively and optimizes the massage result. Sufficient drinking may help to loosen adhesions of the conjunctive tissue (fascia) and to keep muscles, bands etc. elastic.

### 4.1

#### Cleaning and maintenance

- Before cleaning the appliance, make sure that the appliance is switched off and the power supply unit is disconnected. Allow the unit to cool down.
- Clean the device with a slightly damp sponge only. Never use brushes, aggressive cleaning agents, petrol, thinners or alcohol. Rub the unit dry with a clean dry cloth.
- Never immerse the appliance in water to clean it and ensure water or any other fluid cannot get into the appliance.
- Wait until the device is completely dry before using it again.
- Undo any kinks in the cable.
- Take care when rolling up the mains lead to avoid damaging it.
- You should ideally store the device in its original packaging and keep it in a secure, clean, cool and dry place.

### 4.2

#### Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Consult your municipal authority or your dealer for information about disposal.

### 4.3

#### Technical specifications

Name and model	: <b>MEDISANA VibrationBall</b>
Power supply	: Input: 100 - 240 V~, 50/60 Hz, 0.5 A Output: 9V $\approx$ 1000 mA Type of adaptor: ZD012A090100EU
Battery specifications:	7.4V $\approx$ 2200mAh
Charging time	: Approx. 3 hours
Automatic switch-off	: After approx. 15 minutes
Operating conditions	: Only use in dry rooms
Storage conditions	: Clean, dry and cool
Dimensions	: Approx. $\varnothing$ 12 cm
Weight	: Approx. 0.8 kg
Article No.	: 79510
EAN Code	: 40 15588 79510 7

**In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.**

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

**Warranty and repair terms**

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

 **MEDISANA AG**  
Jagenbergstraße 19  
41468 NEUSS  
Germany

The service centre address is shown on the attached leaflet.

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