MEDISANA®

home of wellness

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation.

Read the instruction manual thoroughly.

Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Protection category II

LOT

Lot number

Manufacturer

about the power supply

- Before connecting the device to your power supply, make sure that it is switched off and that the electrical voltage specified on the device's type label matches that of your mains power.
- Keep the power cord and the device itself away from heat, hot surfaces, damp and liquids. Avoid touching the plug or power switch with wet or even damp hands, or when you are standing in water.
- If the device falls into water, do not try to pick it up. Instead pull the plug out immediately.
- Never leave the device unattended when it is plugged into the mains.
- Disconnect the plug from the socket immediately after using the device.
- Never pull on the power cord to disconnect the appliance from the mains. Always pull out the mains plug!

for specific groups of people

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- The surface of the appliance is hot. People who are insensitive to heat must be careful when using the appliance.
- Massages using the handheld massage device are not a substitute for treatments administered by doctors or physiotherapists.
- Do not use this device if you are pregnant.
- People fitted with a pacemaker, artificial joints or electronic implants should in all cases consult a doctor before using the device.
- You should not use this device if you are affected by one or more of the following illnesses and conditions: circulatory disorders, varicose veins, open wounds, bruises, swellings, burns, cracked skin, skin rashes, ulcerated inflammations, phlebitis or thrombosis.

- Do not use the massage device on parts of the body that are numb or insensitive to heat, pressure or pain, or only do so after proper consultation with your doctor.
- Never use the device where there is unexplained pain in the stomach or calves.
- If you feel any pain from use of this device, or find the massage unpleasant, stop the treatment and consult your doctor.

concerning operation

- Check the device and its power cord carefully for any damage before each use. The device should not be used if it is in any way defective.
- Do not use the device if there is any visible damage to the device itself or to the power cord, if the device is not working flawlessly or if it has been dropped or has fallen into water.
- If the power supply cable is damaged, it may only be replaced by **MEDISANA**, an authorised dealer or by qualified personnel. In order to prevent hazards, always send the unit in to the service centre.
- The device is intended for private use only and should not be employed for commercial purposes.
- Use the massage device only for the purpose specified in the instructions for use.
- The guarantee is invalidated if the device is misused.
- While using the red light function, do not place the massager on the same part of the body for too long (max. 1 minute), as this could cause skin rashes or burns.
- Keep the device away from rain and damp. It should not be operated outdoors.
- It is vital to avoid placing, using or storing the device above containers holding liquids, particularly where the device could fall or be knocked into a bathtub or shower tray.
- Do not use the device whilst in the bath or the shower.
- Do not use the device underneath bedding or cushions. Under no circumstances should you cover the device during use. Overheating may lead to burns, electric shock or personal injuries.

GB

1 Safety Information

MEDISANA®
home of wellness

- Do not fall asleep while using the massager on yourself. Using the massager in the same place for a long time can cause complaints and burns.
- Avoid using the device directly on the head, bones or joints, on the front of the throat, on or near the carotid artery, in the genital area or around the kidneys.
- Do not use the device for more than 15 minutes maximum at a time, and allow it to cool down for about 20 minutes before using it again.
- It is best not to use this device directly before retiring - the massage that it delivers has a stimulating effect and may delay sleep.

concerning maintenance and cleaning

- Cleaning and user maintenance shall not be made by children without supervision.
- You the user should not carry out any work on the device, except for cleaning. To avoid possible injury, do not attempt to repair the device yourself. Contact the service point for all repairs.
- Before cleaning the device, switch it off, unplug it from the socket, and allow it to cool. Never immerse the device in water or other liquids.
- Read the information in the section "Cleaning and maintenance".
- Do not repair the device yourself. This would automatically invalidate the guarantee. The service point alone should be charged with any repair work.
- When the device is not in use it should be stored in a clean and dry place, ideally in its original packaging.

2 Useful Information

Thank you very much

Thank you for your confidence in us and congratulations on your purchase!

By purchasing the **HM 855** hand-held massage device, you have acquired a high-quality **MEDISANA** product. To ensure that you are able to obtain the results you are looking for and can enjoy using your **MEDISANA** hand-held massage device **HM 855** for a long time to come, we recommend that you read carefully the following information about the use and care of the device.

2.1 Items supplied and packaging

Please check first of all that the unit is complete and is not damaged in any way. In case of doubt, do not use the appliance and contact your dealer or your service centre.

The following parts are included as standard:

- 1 **MEDISANA** Hand massager with red light **HM 855**
- 1 instructions for use

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

**Please ensure that polybag packing is kept away from reach of children!
Risk of suffocation!**

2.2 How does the HM 855 massager work?

The **MEDISANA HM 855** massager utilizes what is known as vibration massage, whereby the massage head is subjected to high-frequency oscillation, giving rise to a vibrating motion, which is transferred to the parts of the body being massaged.

It also features an optional red light function, emitted through the massage head, which can be activated together with massage. This special form of heat treatment accelerates the positive effects of the massage. When this is activated, the massage head will warm up slightly, but not actually get hot. The unique combination of intensive massage and heat treatment allows the **HM 855** massager to offer effective and relaxing therapy.

3 Operating

Insert the power plug **③** in a designated socket (230 V). Press the function selection switch **②** to turn on the device. The vibration in the massage head **①** is immediately noticeable. Route the device to the body area to be massaged. Do not use pressure; just place it lightly on the body. The massage effect is brought about by the vibration of the unit alone. Simultaneous use of the red light function improves the effectiveness of the massage. Do not concentrate on one side of the body for too long (max. 1 minute); rather change the area of massage frequently. The red light treatment can be switched on by pressing the function selection switch **②** once more, at the same time the red light source illuminates the massage head **①**. Choosing the direction of the massage you can decide on the effect: a massage toward the heart relaxes, a massage away from the heart stimulates. Each session should not be more than 10 - 15 minutes. To ensure positive results, please use the massager regularly. Pressing the switch **②** a third time, the device turns off. After treatment, switch the massage device off, pull out the mains plug and let the device cool down.

GB

4 Miscellaneous / 5 Warranty

MEDISANA®
home of wellness

4 Miscellaneous

4.1 Cleaning and maintenance

Before cleaning the unit, make sure that it is switched off and unplugged from the mains outlet. The device should be cleaned using a soft, slightly dampened cloth. Do not use harsh cleaning agents or stiff brushes. When cleaning, never immerse the unit in water and make sure that no water gets into the unit. Wait until the device is completely dry before using it again.

4.2 Disposal

This product must not be disposed of together with domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Consult your municipal authority or your dealer for information about disposal.

4.3 Technical data

Name und Modell	: MEDISANA Hand-held massage device HM 855		
Power supply	: 220-240V~ 50/60Hz	Power consumption	: approx. 8W
Short operation	: 15 minutes	Storage conditions	: in a cool, dry place
Dimensions	: approx. 28 x 6 x 7 cm	Weight	: approx. 0,3 kg
Item number	: 88297	EAN number	: 40 15588 88297 5

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under www.medisana.com

5 Warranty

Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA AG

Jagenbergstraße 19 • 41468 NEUSS • GERMANY
eMail: info@medisana.de • Internet: www.medisana.de

The service centre address is shown
on the attached leaflet.

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